



JanaBand™

Enhancing safety at the bedside using a novel device to organize cords, tubes, cables, and wires.


STEP 1: Wrap JanaBand™ around bedrail or post and insert notched end into thin slot on opposite end.

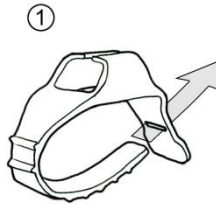

STEP 2: Firmly pull strap through until tight, strap will lock in place.

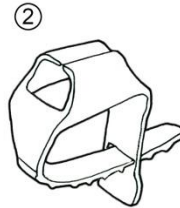

STEP 3: Ready to use! Feed tubes, cables, cords through large clasp on top.

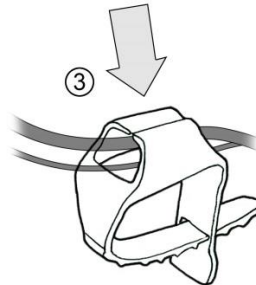

Color Code: Use colors as you like to help you organize.

JanaBand™ is patent pending by JMC Global Technologies I, LP. For more information, please contact Craig Corato at

817-366-4945

craig@janaband.net

www.janaband.net