


# JanaBand®

Enhancing bedside safety, organizing and retaining different size cables, tubes wires and lines at a fixed location.


## **STEP 1**

Determine location and wrap JanaBand® around bedrail or post and insert notched end into thin slot on opposite end:

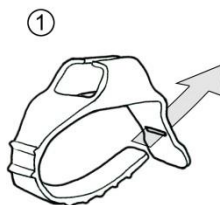


## **STEP 2**

Firmly pull strap through slot until tight, strap will lock in place:

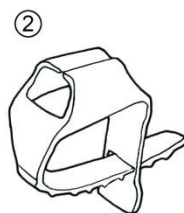

## **STEP 3**

Feed desired tubes, cables, cords through large flap opening on top as shown:

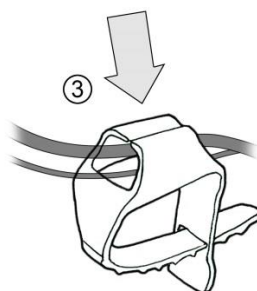

Multiple locations and/or color coded applications are possible.

JanaBand® is patent pending by JMC Global Technologies I, LP. For further questions or additional information, please contact: Craig Corato at 817-366-4945 or write to: [craig@janaband.net](mailto:craig@janaband.net)

**[www.janaband.net](http://www.janaband.net)**